SPORT-ELEC® multisportpro

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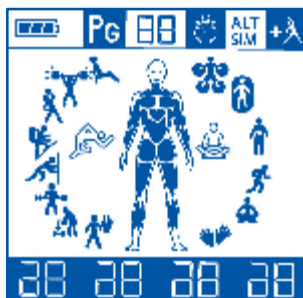
INFO CONSO : 02 32 96 50 50

MULTISPORTPRO (4 CHANNELS)

OVERVIEW OF THE DEVICE (see captioned diagrams on page 3)

- 1. OK key**
Validation after each selection.
- 2. Left key (<)**
Back or select another choice.
- 3. C key (clear)**
Back after validating.
- 4. Right key (>)**
Choice selection key (programs, muscular zones).
- 5. ON/OFF key / Emergency stop (⏻)**
Turns the unit On/Off.
- 6. 4 buttons for power regulation (+ / -)**
+: Increases the power / Level 1 to 10.
-: Decreases the power / Level 10 to 1.
- 7. Connexion module**
- 8. Colour module indicator**
Distinguishes each module for setting of the speed adjustment (+/-).
- 9. USB connection sockets**
- 10. Charging battery connection socket (DC IN jack)**
Can be used to connect the power adapter.
- 11. Locking system slider**
Push the button to the left to operate the device.
Push the button to the right to charge the device.
- 12. Fastener buttons**
Can be used to attach one dual adhesive pad equipped with snap fasteners.

DESCRIPTION OF THE SCREEN

Battery level

Displays the program number chosen

Displays the countdown

ALT Alternate muscles option
(choice between programs to).
Train 2 muscle groups alternately.

SIM Simultaneous muscles option (default choice).
Train 2 muscle groups simultaneously.

Enable/disable warm-up phase

SPORT program

HEALTH program

BEFORE USING THIS UNIT

You have just acquired an effective and powerful professional unit. Take your time to discover how to use it.

- 1 Read the instructions carefully.
- 2 Before using it, charge the unit (pictures 1, 2, 3 page 3).
- 3 Test your unit on your quadriceps using the quick test procedure below.

THERAPEUTIC APPLICATIONS

- **Private:** to strengthen and / or to maintain muscle strength.
- **Physiotherapists:** to ease muscular tensions.
- **In hospital environment:** to prevent muscular atrophy and to treat pain.

OPERATION QUICK TEST

TESTING THE DEVICE ON YOUR QUADRICEPS

- Connect one electrode with snap fasteners to the first module (white).
- Remove the plastic and stick to the skin.
- Start the unit using the key and press the OK key 6 times.
- Increase gradually the power on this channel pressing successively 11 times.
- Once this procedure is done, you can start a program that best suits your needs.

PROGRAMS

SPORT PROGRAMS

Pg 01 SKIN PREPARATION

Lowers the impedance of the skin barrier. Prepares your skin for the currents emitted by the programs.
Duration: 2 min active phases

Pg 02 RESTORE MUSCLE VOLUME

This program prevents and fights loss of muscle volume resulting usually from a lack of use of a limb following a period of immobilisation.
Duration: 40 min active phase + 6 min recovery

Pg 03 REINFORCE MUSCLE VOLUME

Apply this program only on muscles with a normal volume. PG04 and PG05 are used to develop muscle strength.
Duration: 20 min active phase + 6 min recovery

Pg 04 MUSCLE RESISTANCE

The stimulation frequencies aim at increasing the ability to sustain major muscle efforts.
Duration: 30 min active phase + 6 min recovery

Pg 05 MUSCLE ENDURANCE

After using PG04 to acquire the ability to sustain major efforts, use program PG05 to extend high intensity muscle efforts over time by maximum stimulation of slow fibres.
Duration: 30 min active phase + 6 min recovery

Pg 06 MUSCLE FITNESS

Use this program to maintain the level of muscle endurance, resistance, and volume acquired.
Duration: 30 min active phase + 6 min recovery

Pg 07 EXPLOSIVE FORCE

This program is a logical extension of PG03 and PG04. It allows reaching maximum muscle force by stimulating very high speed fibres, with maximum contraction intensity.
Duration: 30 min active phase + 6 min recovery

Pg 08 RECOVERY

Using a frequency range between 0.5 and 8 Hz, this program ensures muscles recover faster through relaxing, endorphin-based effects, and increases blood flow.
Duration: 20 min active phase

HEALTH PROGRAMS

Pg 09 MESSAGE

This program uses frequencies modulated between 3 and 10 Hz for optimum massaging effect.
Duration: 20 min active phase.

Pg 10 MUSCLE RELAXING

The frequency range between 0.25 and 3.5 Hz of this program accelerates muscle relaxing.
Duration: 20 min active phase.

Pg 11 ACTIVE RECOVERY

Use this program immediately after performing intense efforts. All frequencies generated by PG11 maintain muscle work after effort, enhancing blood flow and the release of endorphins. Each of these frequency sequences is alternated with a relaxing effect.
Duration: 20 min active phase.

Pg 12 TENS

This program operates by across-skin nerve stimulation, delivering a range of pulse frequencies in the local area where the electrodes are placed.
Duration: 20 min active phase.

Pg 13 ENDORPHINIC

This program relieves muscle tensions effectively.
Duration: 20 min active phase.

Pg 14 VASCULARIZATION

This program relieves numb legs sensation.
Duration: 26 min active phase.

I. CHARGING THE BATTERY

The device contains a rechargeable battery, immediately following purchase, simply charge it for 4 hours by plugging the unit into the mains power supply.

It is not possible to use the device whilst the battery is charging.

If the device remains unused, charge the battery for 4 hours (maximum) every 4 months.

The device switches itself to stand-by mode if the battery needs charging, this is to prevent the battery to be unloaded.

TO CHARGE THE UNIT (see diagrams 1, 2, 3 page 3)

- ▶ Disconnect the 4 modules cables from the unit.
- ▶ Close the locking system by pushing the button to the right (4 USB locked, jack DC IN open).....>
- ▶ Connect the charger supplied with the unit.

IMPORTANT! Before charging batteries, make sure that the locking system is properly closed.

DISPLAY DURING THE BATTERY CHARGE

- ▶ While charging the battery, the indicator battery displays , , then to show that the battery is charging.
- ▶ Unplug the AC adapter in order to block out the energy supply when the battery is fully charged.
- ▶ When unplugging the AC adapter, the screen switches off.
- ▶ When the battery indicator is empty , it means that the battery is discharged.

II. STARTING THE DEVICE

1 DISPLAY – POWERING THE DEVICE ON

Beware to starting the device, press briefly maximum 1 second the red key in order to switch the device on. Wait a complete switch on of the unit before pressing another key.

All pictograms display and shut off, then all front muscles blink.

If you press permanently, the screen will blink and display all the available programs.

Press the key for switching the device off.

Or for returning to the previous menu, press the key or press briefly the key twice.

2 SELECTING THE MUSCLE ZONE(S) TO STIMULATE

▶ Choosing the 1st zone

The screen displays , meaning you can choose a first muscle zone.

↳ To choose the front muscles, validate with the key: the 1st muscle zone at the bottom blinks: quadriceps.

To choose the back muscles, press the key: the 1st muscle zone at the bottom blinks: gastrocnemius (calves)

To return to the front muscles, press the key.

↳ Press the key to move upwards and scroll the different muscle zones.

To go back, press the key.

↳ After selecting the muscle group, validate using the key

▶ Choosing the 2nd zone

All pictograms blink and the screen displays . The screen proposes to choose a 2nd muscle zone:

↳ To choose a 2nd muscle zone, validate again either the front muscles or the back muscles, validate .

Choose again another muscle zone, then validate key.

Recommendations

The pictograms for upper muscles and for lower muscles are different. Avoid selecting upper muscles and lower muscles, as the unit would give priority to upper muscles, reducing somewhat the yield of work on lower muscles.

3 SELECTING THE PROGRAM

Programs on the left (SPORT programs) blink.

↳ If you choose the SPORT programs , validate .

↳ To choose the HEALTH programs , press the key and validate .

Scroll through the programs by pressing the key:

↳ The screen first displays the 1st lower pictogram while displaying the corresponding program number at the top.

↳ To go back, press the key.

You have now chosen the muscle zone(s) to be stimulated and the program.

4 CHOOSE THE WARM-UP PHASE AND ALT/SIM OPTION

(Only for programs 02 to 07).

- The screen proposes a warming-up phase before starting the active phase of the program.
The WARM-UP PHASE pictogram blinks in the top right corner.
To select it, press **OK**, otherwise press the **<** key and validate **OK** (it disappears from the screen).
- Then the **ALT/SIM** pictogram blinks.
To select **ALT**, press the **<** key, then validate **OK**.
To select **SIM**, press the **>** key, then validate **OK**.
If you have selected the warm-up phase, **SIM** is displayed when starting the program, as the warm-up phase is only in the simultaneous mode.

The pictogram for the program chosen is lit, the muscle zone(s) chosen blink alternately.
The screen is displayed:
 - the battery level,
 - the phase duration (countdown in minutes),
 - the timer pictogram,
 - the **ALT** or **SIM** option and warm-up phase if you have selected them.

5 LAUNCHING THE START - ADJUSTING INTENSITY

You must press the **+** buttons matching the cables to the pads (or to the accessories) for starting a program.

For increasing the intensity level and changing the strength of the stimulation, press 11 times to move to the next level.

The intensity level(s) is/are displayed at the bottom of the screen. Do not start a program if the battery level is low.

The intensity ranges vary during the session according to the program.

Warning! The unit is equipped with a security system that locks the launch of a program if the electrode pads are not placed correctly on the skin.

6 PROGRAM PERFORMANCE

► Use of the **C** (Clear) key

During the program, you can press the **C** key to cancel the ongoing phase.

The next phase starts unless you have reached the end of the program.

In this case, this key allows returning to selecting the 1st muscle zone (as at normal end of program).

► Disconnecting the electrodes during the program

If an electrode is disconnected, the channel concerned is reset to **00** and blinks for 30 sec, before returning to a fixed **00**.

During these 30 sec, the electrode(s) can be reconnected, and power restored to the level required.

7 END OF PROGRAM - STOPPING

At the end of the session, the screen proposes to start a new session by blinking the front muscles to select the 1st muscle zone.

Restart the starting procedure if you wish to continue on another muscle zone.

Otherwise, press the **⏻** key to stop.

The unit switches to standby after 5 minutes idle.

III. USING THE UNIT

GETTING STARTED

- Connect each module to the unit by matching the color indicators (photo A page 4).
Note: You can use the device with 1, 2, 3 or 4 modules.
- Fix the electrodes to the module(s) using the snap fasteners (photo B page 4).

POSITIONING OF THE ELECTRODES

- Remove the plastic backing sheet from the electrodes (photo C page 4).
Moisten them by spreading a few drops of water onto the adhesive gel using your fingertips and wait 2 minutes (photo D page 4).
- Position the electrode(s) onto your body.
Note: The electrodes are for personal use. For hygiene reasons, they must always be used by the same person.

STARTING THE PROGRAM

Start your device by following the "Starting the device" procedure (chapter II).

Contractions should be firm but must never cause any pain.

If the electrodes cause you any discomfort: reduce the power levels, move the electrode(s) and then gradually increase the power. Also ensure that the electrodes are sufficiently moistened.

Now that the settings have been optimized, let your device get to work for the full duration of the program.

END OF PROGRAM

Once the program has finished, switch off the device by pressing the  key, then disconnect the modules.

Gently remove the electrodes from your skin.

Moisten them slightly with a few drops of water and replace them carefully onto their transparent plastic backing sheet.

Keep the adhesive electrodes in a plastic bag in a cool place (for example, a separate compartment in your refrigerator).

IV. RECOMMENDATIONS OF USE

- ▶ The unit is recommended for physiotherapists for functional re-education.
- ▶ You can restart a program's cycle on another muscle zone or change the program while remaining on the same muscle zone, without exceeding however 50 min use per muscle zone and per day.
- ▶ Clean the unit and its cables with a cloth dampened with a disinfecting solution diluted in water at 1/10th.
- ▶ The adhesive electrodes can be rehydrated before and after use with a disinfecting agent.
- ▶ The adhesive electrodes must be repositioned on their support, and stored in a fresh storage between +5°C and +10°C.
- ▶ The adhesive electrodes have a lifetime of about 40 uses, you can order them directly from the online shop www.sport-elec.com or from your reseller.

WARNINGS

- ▶ Keep away from children or people with mental disabilities. The device contains small pieces that can be swallowed.
- ▶ In order to prevent any risk of strangulation, please keep the electrodes wires away from children.
- ▶ The device is meant for personal use. For reasons of hygiene, the electrode should only be used on one person.
- ▶ Do not use the device by driving or by working on a machine.
- ▶ Do not use this electro stimulation unit in a damp environment, a bathroom, a sauna or close to electric machinery connected to mains and earthed, or even piping earthed.
- ▶ Use this device in a clean place (without dust nor dirt...).
- ▶ The simultaneous connection of a patient to a high frequency surgical unit may cause burns at the points of contact of the stimulator's electrodes and the stimulator may be damaged.
- ▶ Operation in close proximity of a short wave therapy unit may cause instability in the stimulator's output power.
- ▶ Do not, under any circumstances, attempt to open or repair the Multisportpro yourself! Only have repairs carried out by the customer service department.

TROUBLE SHOOTING

If your device doesn't work properly, do not use it and contact your reseller.

PRECAUTIONS OF USE

- ▶ Do not use the unit on your face.
- ▶ Do not leave within reach of children.

Ask for advice from your physician:

- ▶ if you have electronic or metallic implants (pins...),
- ▶ in case of skin condition (wounds...).

CONTRA-INDICATIONS

Do not use SPORT-ELEC®:

- ▶ On the heart area.
- ▶ If you carry a pacemaker.
- ▶ If you are pregnant.
- ▶ During first 6 weeks after baby birth.
- ▶ In case of neuromuscular diseases.
- ▶ In case of hemorrhagic diseases.
- ▶ On the face and skull.

RECOMMENDATIONS FOR MAINTAINING AND CLEANING

Clean the unit with a tissue dampened with a disinfecting solution diluted with water at 1/10th.

V. TECHNICAL SPECIFICATIONS

ADHESIVE ELECTRODES

Class I medical device (Directive 93/42/CEE)

Adhesive, hydrophilic and hypoallergenic electrodes

Reference.....EASF-BLACK / EAEXP100x50

Contact area.....67.5 x 47 mm (x2) / 100 x 50 mm (x2)

Estimated life expectancy.....40 uses

Please order new compatible adhesive electrodes on www.sport-elec.com.

AC ADAPTOR

Reference.....MN-A403-E120

Input.....100-240V, 50/60Hz, 0.5A

Output.....5V $\pm$ 0.6A

The adaptor is part of the equipment. Only use the AC adaptor supplied with the device.

Please replace this adaptor should it fail (contact SPORT-ELEC® or order at www.sport-elec.com).

UNIT

Class IIa medical device in accordance with Directive 93/42/CEE

Software version.....E

Type of current.....Dual phase

Number of programs.....8 "SPORT" programs - 6 "HEALTH" programs

Number of program variations (or phases).....344

Number of adjustable independent outputs.....4

Maximum intensity on a 500 Ω load/channel..... 100 mA / 500 Ω / channel

Frequency range.....0.25 to 160 Hz

Pulse width range.....70 to 360 μ s

Power supply.....LiPo Rechargeable battery 3.7V - 1020 mAh

Unit dimensions and weight.....83.5 x 125.5 x 29.61 mm - 340 g

Automatic stop at end of each program.....Yes

Activation safety.....100%

Temperature for stocking / Relative humidity.....0°C to +45°C / 10% to 90%

Temperature for use / Relative humidity.....5-40°C / 20-65%

Caution!..... Read the manual thoroughly

Guarantee of the unit / Lifecycle.....2 years (unit)

CE CE marking concerning medical
0459 (directive 93/42/CEE) appliances.

Device in contact with the patient, excluding the heart area.

RoHS

These units are free of materials likely to have a negative impact on the environment.

IP 22

Protected against the ingress of foreign $\geq$ 12.5 mm and against dripping water when tilted up to 15°.

Caution :
Read the instructions for use.

90mA/1000 Ω

Maximum intensity per channel up to 1 k Ω :
90 mA.

Relative humidity for storage:
from 10 to 90%.

Temperature for storage:
from 0 to 40°C.

Name of the manufacturer bringing the device to the market.

Please contribute to protect the environment by observing the disposal instructions.

The manufacturer respects the environmental law and contributes to the cost of recovery and recycling.

Follow the user manual instructions.

Do not use if you carry a pacemaker.

Do not use if you are pregnant.

This device is intended for domestic use.

Double insulated electrical appliance (> 50V).

VI. GUARANTEE

We provide a 24 months guarantee for products supplied by us, from the date of purchase.

The adhesive electrodes are for personal use. For hygiene reasons, they can therefore neither be returned nor exchanged.

The life expectancy of the adhesive electrodes is estimated at approximately 40 uses.

Never attempt to repair or modify the device yourself or by a repairman not authorized by SPORT-ELEC®. The guarantee becomes null and void if repairs have been carried out by non-authorized persons or if the replacement parts used are not from the same manufacturer.

Similarly, only use compatible accessories manufactured by SPORT-ELEC®.

During the guarantee period we repair, free of charge, manufacturing defects or faulty parts, reserving the right to decide if certain parts need repairing or replacing, or if the device itself must be replaced.

Any damage resulting from improper use (knocks, plugging into a direct current, wrong voltage...) does not fall under the guarantee; normal wear and tear does not affect the proper functioning or value of the device.

The guarantee is only valid if the date of purchase and the stamp and signature of the retailer appear on the guarantee certificate or upon presentation of the till receipt (see "guarantee certificate" at the end of this manual).

No other claims fall under the guarantee, unless otherwise provided by law.

ACCESSORIES GUARANTEE

The electrodes are designed for personal use. For reasons of hygiene, there will be no trade-in nor exchange.

DO NOT FORGET TO ACTIVATE YOUR WARRANTY ONLINE: www.sport-elec.com/activation

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