body control *System* **MULTIPOSITION**

ABDOS - FESSIERS - CUISSES - BRAS

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MULTIFUNCTIONAL BELT
CINTURÓN MULTIFUNCIONES
CINTURA MULTIFUNZIONALE
MULTIFUNCTIONELE GORDEL

SPORT-ELEC®

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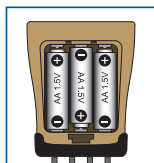
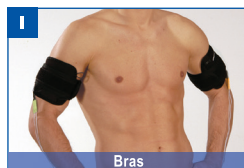
CONNECTION

AJUSTEMENT

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x 3
AA LR6
1.5 V

- NON FOURNIES
- NOT SUPPLIED

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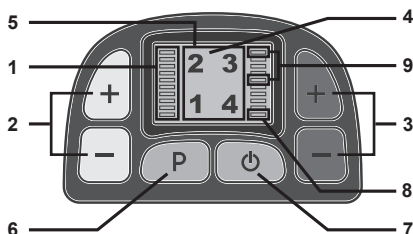
DESCRIPTION OF THE UNIT

1. Indicator of power level
2. Power key (Output A)
+: increase power / level 0 to 64
- : decrease power / level 64 to 0
3. Power key (Output B)
+: increase power / level 0 to 64
- : decrease power / level 64 to 0
4. LCD Display
5. Program indicator
6. Program selection key
7. ON/OFF key
8. Disconnection warning light:
blinks when the belt (or the panty) is disconnected.
9. Battery charge warning light :
the 6th segment blinks : batteries are low
the 10th segment blinks : batteries are used

PROGRAMS

PROG 1 FIRMNESS

Pulse rate = 70Hz
ON time = 2.5 seconds
OFF time = 2.5 seconds
Ramp Up and Down = 2 seconds
Program timer = 20 minutes

PROG 2 MUSCLE CARE

Pulse rate = 60 Hz
ON time = 3.5 seconds
OFF time = 3.5 seconds
Ramp Up and Down = 2 seconds
Program timer = 26 minutes

PROG 3 MUSCLE FORCE

Pulse rate = 50 Hz
ON time = 4.5 seconds
OFF time = 4.5 seconds
Ramp Up and Down = 2 seconds
Program timer = 30 minutes

PROG 4 MASSAGE

Pulse rate = 50 Hz
ON time = 5.5 seconds
OFF time = 5.5 seconds
Ramp Up and Down = 2 seconds
Program timer = 40 minutes

"ACHIEVING AN ADEQUATE MUSCLE CONTRACTION"

The most suitable effect is reached when you detect that the sensitivity of your muscle responds to the stimulation coming from the BODY CONTROL SYSTEM device, so that :

- You can feel the contraction
 - You can see the contraction
 - The contraction is palpable
- In any case you must not feel a discomfort

CUSTOMER SUPPORT: 00 33 2 32 96 50 50

INTRODUCTION

You have recently acquired the electro-musculature device with accessories, designed, produced and manufactured by SPORT-ELEC® INSTITUT (France). We thank you for your purchase and suggest that you read attentively the following paragraphs for an optimal use of the device. SPORT-ELEC® is specifically designed to enhance the abdominal area and the buttocks.

GENERALITIES

BODY CONTROL SYSTEM (BSC) is conceived for women and men who care about their appearance.

BCS is a device which uses electrical stimulation through the "contraction-release" effect on muscles that you want to work.

The aim of the BCS is to refine your silhouette by acting on the muscular masses (waist, Abs, butt muscles, thighs, and legs).

BCS contains 4 programs for a more particular development of certain muscles (pectoral, abdominal, biceps...) resulting from SPORT-ELEC® research. The frequencies and impulses width of each program have been defined to meet perfectly everybody's requirements :

- Work the muscle gently and facilitate its recovery after an effort
- Improve the firmness of tissues
- Reshape the silhouette

In conclusion, according to our activity, some muscles are more or less developed, the others atrophy on the contrary.

Our body contains more than 600 muscles. The shape and harmony of our body depends on these muscles.

BCS allows activating the set of muscles without any effort from you. Its daily use will restore vigour of your muscle structure.

THERAPEUTIC APPLICATIONS

- Private: to strengthen and / or to maintain muscle strength.
- Physiotherapists: to ease muscular tensions.
- In hospital environment: to prevent muscular atrophy and to treat pain.

WARNING

READ THE FOLLOWING INFORMATION BEFORE USE :

Prior to use, check the following points :

- ▶ The device will stop if you try to start a program without positioning the belt on the body on first.
- ▶ The device must be fully connected to all the 4 connecting plugs of the accessory.
- ▶ Adjust accessories carefully to ensure contact between electrodes and the skin.

CAUTION:

The generator is provided with a safety system to avoid incorrect handling of the device by users. If you move with accessories attached and a partial loss of contact between electrodes and the skin occurs, the device will stop (safety stop).

CONTRA-INDICATIONS

Do not use SPORT-ELEC® :

- ▶ On the heart area.
- ▶ If you carry a pacemaker or suffer from any other heart problem.
- ▶ If you have any form of an abdominal implant. The safety has not been established for the use of electronic muscle stimulators over abdominal implants.
- ▶ If you are pregnant.
- ▶ During first 6 weeks after baby birth.
- ▶ During first 3 months after having a surgery (like caesarean section...) and consult your doctor before using the device.
- ▶ If you have cancer, as the effects of electronic stimulation of cancerous tissue are unknown.
- ▶ In case of neuromuscular diseases.
- ▶ In case of haemorrhagic diseases.

I. INSERTION OF BATTERIES

Check that the device is switched off.

Open the "batteries" compartment, insert three 1,5 V AA LR6 alkaline batteries into the compartment then close the case.

Comply with + and - indications (photo II - A page 2).

Do not mix used and good batteries.

For an optimal use of your device, it is advised to use good quality batteries.

II. STARTING THE UNIT

Your electro-stimulation device is an appliance at the state-of-the-art technically, its design is based on standards EN 60601-1 and -2-10, it is foreseen for improving tone of your muscles, and not for measurement or diagnosis.

The interface "man machine" is simplified and realized by means of numbers and function keys.

From 1 to 4, the number in black indicates you the program chosen by pressing the "P" key.

Two graduated scales (1 by output) indicate you 64 power levels.

You also find 6 order keys (scheme, page 10) :

- ▶ 1 ON/OFF "O" - a short press
- ▶ 1 "P" to select the programs - a short press
- ▶ 2 "+/-" keys per output, in each side of the LCD display, to adjust the intensity of the stimulation.
A long (1 sec.) press on the "-" key allows to return the power level rapidly to zero.

1 FUNCTION 1 - "ON/OFF" SWITCH

- ▶ A short press on the "O" key puts the device under on or off power.
- ▶ Every time under power, the device makes an auto-test on 11 essential parameters at the level of the safety of its operation.
During the auto-test segments and pictograms are lit during 0,5 second.
- ▶ If an internal defect in the device is detected, all the pictograms flash. Send back the device to the After Sale Service.
- ▶ The auto-test does not take into consideration the used batteries. When the batteries are used the autotest cannot be performed.
- ▶ One long press (over 3 sec) on the "O" key stirrs up the auto-test.
In this case, stop the device by pressing the "O" key and then press shortly the "O" key again.

2 FUNCTION 2 - CHOICE OF A PROGRAM "P"

Just make one or several presses on the "P" key, the programs ravel.
The programs are announced by numbers from 1 to 4.
Once the program is chosen, the program number is shown on the display. (see page 10).
Once the program is chosen, you can start it by increasing only one of the both "+" keys.
During the program the "P" key is inactive.

3 FUNCTION 3- END OF PROGRAM - STOP

- ▶ At the end of the program the device stops automatically.
- ▶ To stop the device during a program, you just have to turn 2 power keys to zero or to press the "O" key.
- ▶ In case of emergency, press the "O" key.

4 FUNCTION 4 - ADJUSTMENT THE POWER

You have 64 levels from 0 to 64 to adjust the electro-stimulation power.
Between two segments, there are 7 power levels or 4 short press on "+" key.

5 FUNCTION 5 - BATTERIES

- ▶ If the batteries are "good", there is no blinking.
- ▶ If the batteries are "weak", two segments (the 6th segment) blink (scheme page 10). Your device still works.
- ▶ If the batteries are "used", two segments (the 10th segment) blink (scheme page 10).
- ▶ If your device stops, even during a program: change the batteries.

6 FUNCTION 6 - DISCONNECTION

Your device is endowed with an accessory disconnecting detector.
If there is no contact between the skin and the accessory, the power is stopped on the concerned output (channel A or B).
You are warned of the disconnection by blinking of the 1st segment during 30 sec., after 30 sec. the blinking stops (diagram, page 10).
If no accessory is in contact with the body, the power cannot be emitted.

III. USE THE MULTIFUNCTIONAL BELT

The multifunction belt can be used whole, or separated into 2 to form 2 half-belts.
The full belt must be used on the stomach, buttocks, lower back areas.
Once unfastened, the 2 "half-belts" can be used on the arms, thighs, or calf.
The multifunction belt has built-in electrodes on which the SPORT-ELEC® cream is applied.
This cream enhances contact between the electrodes and the skin.
Its massaging and electro conducting properties improve comfort of use and enhance stimulation.
The cream can also be used for body massage.
If you want to buy the SPORT-ELEC® cream, please order directly by going on www.sport-elec.com, or contact your local supplier.

WARNING!

- ▶ Never use the belt without the cream provided.
- ▶ Do not use any cream other than the one provided by SPORT-ELEC® INSTITUT.

SAFETY PRECAUTIONS FOR CREAM USE

- In case of allergy, stop applying the cream.
- Do not swallow.
- Do not leave with the range children.
- In case of contact with eyes rinse with water.

PRESENTATION OF THE BELT :

- **Front side (photo A page 2) :** 1 = Fasteners - 2 = Wire storage - 3 = Connection terminals - 4 = Scratch strip
- **Electrodes side (photo B page 2) :** 4 built-in electrodes

1 STEP 1: CONNECTING AND FITTING THE BELT.

- ▶ Check your SPORT-ELEC ® unit is stopped.
- ▶ Take the connectors of your device and connect them to the belt connectors per groups of two (photo C page 2).
- ▶ Apply a thick layer of electro-conductive massaging cream onto the electrodes built into the belt (domed part - photo D page 2)
- ▶ Place the belt or 2 half-belts onto the zone(s) to be stimulated, and adjust them using "scratch" strips to ensure the built-in electrodes are properly in contact with the skin (photos page 2 : G = stomach, H = back, I = arms, J = thighs, K = calf)
- ▶ Move the belt slightly left or right (with device powered off) to feel the contractions on the abdominal area.
- ▶ If an electrode hurts or if you feel any tingling, stop the device, and check the electrode is properly smeared with cream.

TIP :

- ▶ Fitting the full belt (photos E page 2) : To position the full belt at best, cross the 2 strips to flip the 2 ends on the outer scratch part.
- ▶ Fitting the half-belt on the arm (photos 4, B page 4) : Adjust at the belt mid-size, according to the outline of your arm before positioning it, then start over if necessary to ensure proper adjustment.

2 STEP 2 : STARTING THE BELT FUNCTION

Once your unit is connected, install yourself comfortably and start your unit using the procedure "Starting the unit" detailed previously in paragraph II.

3 STEP 3 : STOPPING THE BELT FUNCTION

Once the program is finished, disconnect the device from the belt.

Remove the excess cream on the built-in electrodes using a wipe dampened with water, then dry them.

IV. RECOMMENDATION FOR USE**1 PRECAUTIONS**

- ▶ Some persons may experience skin irritation or hypersensitivity due to the electrical stimulation or contact cream.
- ▶ Keep away from children or people with mental disabilities. The device contains small pieces that can be swallowed.
- ▶ Use this device only with the leads, electrodes, and accessories recommended by the manufacturer.
- ▶ Do not use portable electrical stimulators while driving, operating machinery, or during any activity in which electrical stimulation can put the user at risk of injury.

Use with caution in the following circumstances :

- ▶ persons with suspected or diagnosed epilepsy
- ▶ when there is a tendency to bleed internally following an injury of fracture
- ▶ following recent surgical procedures when stimulation may disrupt the healing process
- ▶ over areas of skin that lack normal sensation

GET YOUR DOCTOR'S ADVICE IF :

- ▶ You have back troubles. You should also ensure the intensity is kept low.
- ▶ You have any serious illness or injury not mentioned in this guide.
- ▶ You have recently undergone a surgical procedure.
- ▶ You take insulin for diabetes.

TIPS EXPLANATIONS FOR SMALL SIZES :

- ▶ If the belt overlaps with electrodes on the body, fold the overlapping part of the belt by means of scratch sections.

CONTACT SPORT-ELEC® INSTITUT OR AN AUTHORIZED PROVIDER IF :

- ▶ You experience any irritation, skin reaction, hypersensitivity or other adverse reaction. However, please note that some reddening of the skin is quite normal under the belt during and for a short time after exercise.
- ▶ An effective treatment should not cause undue discomfort. SPORT-ELEC® INSTITUT will not accept responsibility if the guidelines and instructions supplied with this unit are not followed.
- ▶ If you are in any doubt about using the device for any reason, please consult your doctor before use.

RECOMMENDATIONS OF USE

- You can restart a program's cycle on another muscle zone.
- For optimum use, refer to the diagrams shown in this manual.
- Clean the unit and its cables with a cloth dampened with a disinfecting solution diluted in water at 1/10th.
- If the device is not likely to be used for some time, the batteries should be removed.

TROUBLE SHOOTING

- If your device doesn't work properly, do not use it and contact your retailer.

⚠️ RESTRICTIONS OF USE ⚠️

To benefit fully of your electrostimulation session, please pay attention to the following guidelines :

- The belt must not be connected to other objects.
- Do not touch inside the belt while the BODY CONTROL SYSTEM is switched on.
- Do not use while driving or operating machinery.
- Do not use in wet environment.
- Use this device in a clean place (without dust nor dirt.).
- Keep your unit at distance from heat source.
- Keep your unit at distance from short-wave emitting equipment or high frequency surgical devices.
- To reposition the belt during a session :
Always stop the program currently running, unfasten the belt and refasten it after the belt's position has been adjusted.
- After strenuous exercise or exertion :
Always use a lower intensity to avoid muscle fatigue.

2 WARNINGS

The long-term effects of electrical stimulation are unknown.

Wait before using your BODY CONTROL SYSTEM:

- When applying the belt, always remember to place the belt only on the abdomen and as indicated in this manual.
- Do not apply stimulation on the neck. Severe spasm of the muscles may occur and the contractions may be strong enough to close the airway or cause difficulty in breathing.
- Stimulation over the neck could also have adverse effects on the heart rhythm or blood pressure.
- Do not apply stimulation across the chest because the introduction of electrical current into the chest may cause rhythm disturbances to the heart, which could be lethal.
- The effects of stimulation on the brain are unknown. Therefore, do not apply stimulation across the head and do not place electrodes on the opposite side of the head.
- Apply stimulation only to normal, intact, clean skin. Do not apply stimulation over open wounds or over swollen, infected, or inflamed areas or skin eruptions, e.g. phlebitis, thrombophlebitis, varicose veins, etc.
- Do not apply stimulation in the bath or under the shower.
- Do not use this device while sleeping.
- If you have an IUD contraceptive device, do not practice electro-stimulation.
The safety of use of this device when used with an IUD has not been established

WARNING :

- Use only on those parts of the body for which the accessories were designed.
- Functions only on batteries
- Not to connect to another source of power

RECOMMENDATIONS

Please apply regulations enforced in your country when disposing your product at its end of life.

Therefore, please discard it in a location planned to this effect, ensuring it is disposed in an environment-friendly manner.

ADVERSE REACTIONS

Skin irritation and burns beneath the electrodes have been reported with the use of electrical stimulators applied to the skin.

CLEANING & MAINTENANCE CONDITIONS

- **BELT** : No washing - No solvent - No drying - No ironing - No laundering - Clean the belt using a cloth dampened with a detergent, then wipe with a dry cloth - Clean the electrodes with a mix of water/hypoallergenic detergent, then rinse thoroughly.
- **UNIT** : Clean the unit with a cloth soaked in with disinfectant (Mercryl type) diluted with water at 1/10th.

V. TECHNICAL SPECIFICATIONS

1 UNIT:

CE	EC declaration of conformity
IEC standards for the safety and effectiveness of medical electrical equipment	: EN60601-1 / EN60601-2-10
Software version	1
Type of current	I_{T1} : Dual phase rectangular and symetric
Number of programmes	4 programs
Number of adjustable independent outputs	2
Current	From 0 to 60mA from 0 to 60Vpp- from 0 to 1000 Ω
Frequency range / Pulse width range	50 - 70 Hz / 200 μ s
Power	3x 1,5 V alkaline batteries type LR6/AA (not provided)
Unit dimensions and weight	69 x 43 x 87 (h) mm - 106 g
Software version	BCS - B
Automatic stop at end of each program	Yes - about 4 min 30 sec
Activation safety	100%
Detection absence of electrodes or of contact	yes detection independent on each of the 2 outputs
Indicator of power	bar graph - LCD display
Indicator of weak and/or defective battery	yes - LCD display
Indicator absence contact belt	yes - LCD display
Indicator of the programs	LCD display : prog 1-2-3-4
Temperature of storage / Relative humidity of storage	0°C to + 45°C / 10% to 90%
Temperature of use / Relative humidity of use	+ 5°C to + 45°C / 20% to 65%
Connection device - belt	wires - plugs $\varnothing$ 2 mm male
Caution	 Read the manuel thoroughly
Guarantee / Lifecycle	2 years

Device in contact with the patient, excluding the heart muscles.

RoHS (2002/95/EC) The unit are free of materials likely to have a negative impact on the environment.

IP22 Protected against the ingress of foreign object ≥ 12.5 mm & against dripping water when tilted up to 15°.

2 BELT:

Reference	WA15117/4
Size (unisex)	from 34 to 60
Length of the unfolded belt	150 cm (2 x 75 cm)
Fabric	polyester
Built-in electrodes	Carbonated silicon
Estimated lifetime	around 10 months


3 ELECTRO CONDUCTIVE CREAM

Adhesive, hydrophilic and hypoallergenic electrodes	
Reference	VC 57B/53-148
Nominal fill	75 ml / 2.54 fl Oz
Class I medical device (Directive 93/42/CEE)	
Please order new electroconductive cream on sport-elec.com.	

VI. GUARANTEE :

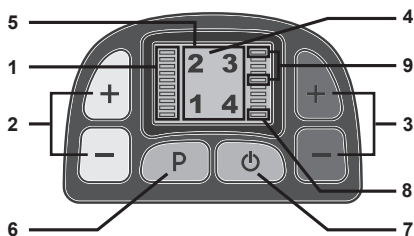- ▶ We provide a 24 month guarantee for products supplied by us, from the date of purchase.
- ▶ The belt and the contact cream are for personal use. For hygiene reasons, they can therefore neither be returned nor exchanged. The life expectancy of the built-in electrodes is estimated at approximately 10 months.
- ▶ Never attempt to repair or modify the device yourself, or by a repairman not authorized by SPORT-ELEC®. The guarantee becomes null and void if repairs have been carried out by non-authorized persons or if the replacement parts used are not from the same manufacturer.
- ▶ Similarly, only use BCS-compatible accessories manufactured by SPORT-ELEC®.
- ▶ During the guarantee period we repair, free of charge, manufacturing defects or faulty parts, reserving the right to decide if certain parts need repairing or replacing, or if the device itself must be replaced.
- ▶ Any damage resulting from improper use (knocks, plugging into a direct current, wrong voltage, ...) does not fall under the guarantee; normal wear and tear does not affect the proper functioning or value of the device.
- ▶ The guarantee is only valid if the date of purchase and the stamp and signature of the retailer appear on the guarantee certificate, or upon presentation of the till receipt. (See «guarantee certificate» at the end of this manual.
- ▶ No other claims fall under the guarantee, unless otherwise provided by law.

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