multisportpro

PRECISION

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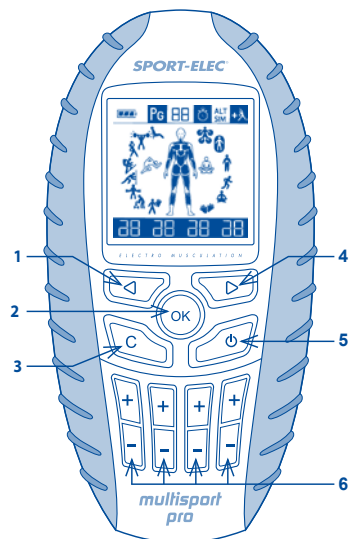
0459 (directive 93/42/CEE)


Made in P.R.C.

MULTISPORTPRO (4 CHANNELS) DESCRIPTION OF THE UNIT

- 1 Left key : Back or select another choice
- 2 OK key : Validation after each selection
- 3 C key (clear) : Back after validating
- 4 Right key : Choice selection key (programs, muscular zones.)
- 5 ON/OFF key : Turns the unit On and Off / emergency stop
- 6 4 Power key :
+ : Increase power / level 1 to 30
- : Decrease power / level 30 to 1

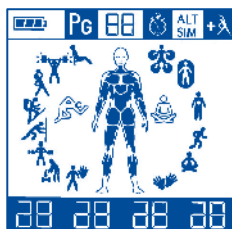DESCRIPTION OF THE SCREEN

- Battery level
- Displays the program number chosen
- Displays the countdown
- ALT**
Alternate muscles option (choice between programs Pg01 to Pg08).
Train 2 muscle groups alternately
- SIM**
Simultaneous muscles option (default choice). Train 2 muscle groups simultaneously
- Enable/disable warm-up phase
- Sport program
- Health program

PROGRAMS

SPORT PROGRAMS

Pg 01

SKIN PREPARATION

Lowers the impedance of the skin barrier. Prepares your skin for the currents emitted by the programs.
Duration : 2 min active phases

Pg 02

RESTORE MUSCLE VOLUME

This program prevents and fights loss of muscle volume resulting usually from a lack of use of a limb following a period of immobilisation.
Duration : 40 min active phase + 6 min recovery

Pg 03

REINFORCE MUSCLE VOLUME

Apply this program only on muscles with a normal volume. PG04 and PG05 are used to develop muscle strength.
Duration : 20 min active phase + 6 min recovery

Pg 04

MUSCLE RESISTANCE

The stimulation frequencies aim at increasing the ability to sustain major muscle efforts.
Duration : 30 min active phase + 6 min recovery

Pg 05

MUSCLE ENDURANCE

After using PG04 to acquire the ability to sustain major efforts, use program PG05 to extend high intensity muscle efforts over time by maximum stimulation of slow fibres.
Duration : 30 min active phase + 6 min recovery

Pg 06

MUSCLE FITNESS

Use this program to maintain the level of muscle endurance, resistance, and volume acquired.
Duration : 30 min active phase + 6 min recovery

Pg 07

EXPLOSIVE FORCE

This program is a logical extension of PG03 and PG04. It allows reaching maximum muscle force by stimulating very high speed fibres, with maximum contraction intensity.
Duration : 30 min active phase + 6 min recovery

Pg 08

RECOVERY

Using a frequency range between 0.5 and 8 Hz, this program ensures muscles recover faster through relaxing, endorphin-based effects, and increases blood flow.
Duration : 20 min active phase

HEALTH PROGRAMS

Pg 09

MASSAGE

This program uses frequencies modulated between 3 and 10 Hz for optimum massaging effect.
Duration : 20 min active phase.

Pg 10

MUSCLE RELAXING

The frequency range between 0.25 and 3.5 Hz of this program accelerates muscle relaxing.
Duration : 20 min active phase

Pg 11

ACTIVE RECOVERY

Use this program immediately after performing intense efforts. All frequencies generated by PG11 maintain muscle work after effort, enhancing blood flow and the release of endorphins. Each of these frequency sequences is alternated with a relaxing effect.
Duration : 20 min active phase

Pg 12

TENS

This program operates by across-skin nerve stimulation, delivering a range of pulse frequencies in the local area where the electrodes are placed.
Duration : 20 min active phase

Pg 13

ENDORPHINIC

This program relieves muscle tensions effectively.
Duration : 20 min active phase

Pg 14

VASCULARIZATION

This program relieves numb legs sensation.
Duration : 26 min active phase

* BEFORE USING THIS UNIT

You have just acquired an effective and powerful professional unit.

Take your time to discover how to use it.

- 1 - Read the instructions carefully
- 2 - Before using it, charge the unit (photo I. E, page 2)

* OPERATION QUICK TEST

TESTING YOUR UNIT ON YOUR QUADRICEPS

- Take the electrodes and lightly moisten them with a few drops of water (photo I. A, page 2).
- Connect it to the first channel (match the colours of the connectors, 2 by 2).
- Start the unit using the  key and press the "OK" button 6 times.
- Increase power on this channel gradually.
- Once this procedure is done, you can start a program that best suits your needs.

* THERAPEUTIC APPLICATIONS

- **Private:** to strengthen and / or to maintain muscle strength.
- **Physiotherapists:** to ease muscular tensions.
- **In hospital environment:** to prevent muscular atrophy and to treat pain.

I. CHARGING THE BATTERIES

The unit contains rechargeable batteries. Before using it, charge it during 12 hours by connecting it to mains. Do not use this unit while batteries are charging. When it is not used, charge the batteries for 12 hours every 4 months. When the batteries are empty, the unit switches to standby automatically, protecting them against untimely discharging. To charge the unit, connect the charger supplied with the unit (photo I. E, page 2)

BATTERY CHARGE DISPLAY

When the battery is charging, the battery indicator displays , , then  to show the current charging status. The charging time countdown starts, from 12 to 00. Once charged, the number 00 appears in the left angle of the timer pictogram. The battery pictogram displays . When the charging is complete, unplug the adapter from the electrical outlet to insulate electric source. When the charger is disconnected, the screen switches off. When the battery indicator is empty , this means that it is discharged.

II. STARTING THE UNIT

STEP 1: DISPLAY - POWER ON

Beware while starting the device (red button )
Push briefly (maximum 1 second) on .
All pictograms appear and blink for 3 seconds, then the front muscles blink.
If you make a long push, your screen flashes and indicates you the totality of the pictograms which it contains.
To stop the device, push briefly on .
Or to return in the menu made 2 brief pushes on .

STEP 2: SELECTING THE MUSCLE ZONE(S) TO STIMULATE

Choosing the 1st zone :

The screen displays , meaning you can choose a first muscle zone.
- To choose the front muscles, validate with the "OK" key.
- To choose the back muscles, press the  key.
- To return to the front muscles, press the  key.
If you have chosen the front muscles : the 1st muscle zone at the bottom blinks: quadriceps.
If you have chosen the back muscles : the 1st muscle zone at the bottom blinks: gastrocnemius (calves)
Press the  key to move upwards and scroll the different muscle zones.
To go back, press the  key.
After selecting the muscle group, validate using th "OK" key

Choosing the 2nd zone :

All pictograms blink and the screen displays .
The screen proposes to choose a 2nd muscle zone :
To choose a 2nd muscle zone, validate again either the front muscles or the back muscles, validate "OK".
Choose again another muscle zone, then validate "OK".
If you wish one muscle zone only, press the  key and go to step 3.
The muscle zone(s) selected is (are) displayed.

STEP 3: SELECTING THE PROGRAM

Programs on the left ( SPORT programs) blink.
If you choose the  SPORT programs, validate "OK".
To choose the  HEALTH programs, press the  key and validate "OK".
Scroll through the programs by pressing the  key.
The screen first displays the 1st lower pictogram while displaying the corresponding program number at the top.
To go back, press the  key.
You have now chosen the muscle zone(s) to be stimulated, and the program.

STEP 4: CHOOSE THE "WARM-UP PHASE" AND ALT SIM OPTION

(Only for programs  P6 to  P7).
The screen proposes a warming-up phase before starting the active phase of the program.
The  WARM-UP PHASE pictogram blinks in the top right corner.
To select it, press "OK", otherwise press the  key, and validate "OK" (it disappears from the screen).
Then the  pictogram blinks.
To select "ALT", press the  key, to select "SIM", press the  key, then validate "OK".
If you have selected the warm-up phase, "SIM" is displayed when starting the program, as the warm-up phase is only in the simultaneous mode.
The screen is displayed :
- the battery level,
- phase duration (countdown in minutes),
- timer pictogram,
- the "ALT" or "SIM" option and warm-up phase if you have selected them.
The pictogram for the program chosen is lit, the muscle zone(s) chosen blink alternately.

STEP 5: LAUNCHING THE START - ADJUSTING INTENSITY

Then, press the "+" buttons of the intensity keys matching the wires connected to your electrodes to start the program.
If the battery is low, do not start the program.
The power level(s) are displayed at the bottom of the screen. The intensities vary during the session according to the program's content.
A long press on the "-" key returns the power rapidly to 0.

STEP 6: PROGRAM PERFORMANCE

Use of the "C" (Clear) key:

During the program, press the "C" key during a phase to set its time to zero.
The next phase starts unless you have reached the end of the program.
In this case, this key allows returning to selecting the 1st muscle zone (as at normal end of program).

Disconnecting the electrodes during the program :

If an electrode is disconnected, the channel concerned is reset to 00 and blinks for 30 sec, before returning to a fixed 00.
During these 30 sec, the electrode(s) can be reconnected, and power restored to the level required.

STEP 7: END OF PROGRAM - STOPPING

At the end of the session, the screen proposes to start a new session by blinking the front muscles to select the 1st muscle zone.
Restart the starting procedure if you wish to continue on another muscle zone.
Otherwise, press the  key to stop.
The unit switches to standby after 5 minutes idle.

Recommendations:

The pictograms for upper muscles and for lower muscles are different. Avoid selecting upper muscles and lower muscles, as the unit would give priority to upper muscles, reducing somewhat the yield of work on lower muscles.

III. USING THE UNIT

STEP 1

Take the electrodes and remove carefully their protection, dampen them one by one with water on your fingertips on the adhesive gel, and wait for 2 minutes (diagram I. A, page 2).

STEP 2

Fit the electrodes on your body as shown in diagram II. A or B or C, page 2: Positions are shown for information only

Important: For optimal use, carefully read each page and look at the examples of use (photos II, page 2).

Examples of use (photos II, page 2)		
Ⓑ = connector channel B (middle blue); Ⓒ = connector channel C (light blue); SUP = upper limbs; INF = lower limbs		
A photo - Chest	B photo - Shoulders	C photo - Trapezius
1 - Large chest ⚠ Do not place the electrodes close to the heart	2 - Medium deltoid 3 - Posterior 4 - Anterior	9 - Upper trapezius 10 - Medium trapezius

STEP 3

Take connector (CEV8) (photo I. B, page 2) and connect it:

1 - to the electrodes (photo I. C, page 2)

2 - then to the unit (photo I. D, page 2).

STEP 4

The unit is connected to the electrodes through the connector, composed of 2 groups of 4 x 1 m. long wires completed by 2 groups of 4 connection plugs.

It is possible to hang your MULTISPORT PRO to your clothing using the clip.

STEP 5

Start your unit following the "Starting the unit" procedure described in paragraph II.

Contractions should be firm but never painful. If you feel an unpleasant sensation around the electrodes: bring the power-on key back to minimum, move the electrode(s) and gradually increase the power.

Once the adjustments have been optimised in this way, leave your appliance to work for the full duration of the program.

STEP 6

Once the program is complete, disconnect the electrodes from the (CEV8) connector, then the latter from the unit.

Then pull off the electrodes gently, dampen them slightly with a few drops of water, then return them carefully onto their support. Store the adhesive electrodes in a fresh location, in a plastic bag (e.g. vegetable container in the refrigerator).

IV. RECOMMENDATIONS OF USE

The unit is recommended for physiotherapists for functional re-education.

- You can restart a program's cycle on another muscle zone or change the program while remaining on the same muscle zone, without exceeding however 40 min use per muscle zone and per day.
- For optimum use, refer to the diagrams shown in this leaflet.
- Clean the unit and its cables with a cloth dampened with a disinfecting solution.
- The adhesive electrodes can be rehydrated before and after use with a disinfecting agent (e.g. Hexamedine).
- The adhesive electrodes must be repositioned on their support, and stored in a fresh storage between +5°C and +10°C.
- The adhesive electrodes have a lifetime of about 40 uses, you can order them directly from the online shop www.sport-elec.com, or from your reseller.

PRECAUTIONS OF USE

- Do not use the unit on your face
- Do not leave within reach of children

Ask for advice from your physician:

- if you have electronic or metallic implants (pins...)
- in case of skin condition (wounds...)

WARNINGS

- Keep away from children or people with mental disabilities. The device contains small pieces that can be swallowed.
- In order to prevent any risk of strangulation, please keep the electrodes wires away from children.
- If the device is not likely to be used for some time, the batteries should be removed.
- Multisportpro is meant for personal use. For reasons of hygiene, the electrode should only be used on one person.
- Do not use the device by driving or by working on a machine.
- Do not use this electro stimulation unit in a damp environment, a bathroom, a sauna, or close to electric machinery connected to mains and earthed, or even piping earthed.
- Use this device in a clean place (without dust nor dirt...).
- The simultaneous connection of a patient to a high frequency surgical unit may cause burns at the points of contact of the stimulator's electrodes, and the stimulator may be damaged.
- Operation in close proximity of a short wave therapy unit may cause instability in the stimulator's output power.
- Do not, under any circumstances, attempt to open or repair the Multisportpro yourself! Only have repairs carried out by the customer service department.

TROUBLE SHOOTING

If your device doesn't work properly, do not use it and contact your retailer.

RECOMMENDATIONS

Please apply regulations enforced in your country when disposing your product at its end of life. Therefore, please discard it in a location planned to this effect, ensuring it is disposed in an environment-friendly manner.

CONTRA-INDICATIONS

Do not use SPORT-ELEC®:

- On the heart area
- If you carry a pacemaker
- If you are pregnant
- During first 6 weeks after baby birth
- In case of neuromuscular diseases
- In case of hemorrhagic diseases
- On the face and skull

CARE AND CLEANING INSTRUCTIONS

- Do not expose to bleach.
- Clean the unit with a tissue dampened with a disinfecting solution diluted.
- Keep the electrodes in a dry and cool place. After use, moisten the adhesive gel surface with a few drops of water and hold two minutes.

CUSTOMER SUPPORT: +33 (0)2 32 96 50 50

V. TECHNICAL SPECIFICATIONS

ELECTRODES

Electrodes Adhesive, hydrophilic and hypoallergenic
Size 89 x 50 mm
Estimated lifetime 40 uses
Class I medical device in accordance with Directive 93/42/CEE

ELECTRODES WARRANTY CONDITIONS

The electrodes are for a single person use only. For hygiene reasons, they cannot be returned nor replaced.

UNIT:

Class IIa medical device in accordance with Directive 93/42/CEE

Software version E
Type of current Dual phase
Number of programs 8 "SPORT" programs - 6 "HEALTH" programs
Number of program variations (or phases) 344
Number of adjustable independent outputs 4
Maximum intensity on a 500 Ω load/channel 100 mA / 500 Ω / channel

Frequency range.....	0.25 to 160 Hz
Pulse width range.....	70 to 360 µs
Power supply.....	4.8V - 1300 mAh - Ni-MH Rechargeable battery
Mains supply.....	THA239025 : 230VAC 50Hz 100mA - 9VAC 250mA RSS1002-055110-W2B-B: 100-240VAC 50/60Hz 0.18A - 12VDC 250mA The adapter is part of the equipment. Only use mains power after reticulation by the power unit.
Unit dimensions and weight.....	80 x 160 x 35 mm - 340 gr
Automatic stop at end of each program.....	Yes
Activation safety.....	100%
Temperature for stocking / Relative humidity.....	0°C to + 45°C / 10% to 90%
Temperature for use / Relative humidity.....	5-40°C / 20-65%
Caution !.....	⚠ Read the manual thoroughly
Guarantee of the unit / Lifecycle.....	2 years

STANDARDIZED SYMBOLS / PICTOGRMS

CE CE marking concerning medical appliances.
0459 (directive 93/42/CEE)

Device in contact with the patient, excluding the heart area.

RoHS These units are free of materials likely to have a negative impact on the environment.

IP 22 Protected against the ingress of foreign object ≥12.5 mm & against dripping water when tilted up to 15°.

Caution :
Read the instructions for use.

Maximum intensity per channel up to 1kΩ :
90mA.

Relative humidity for stocking :
from 10 to 90%.

Temperature for stocking :
from 0 to 40°C.

Name of the manufacturer bringing the device to the market.

Please contribute to protect the environment by observing the disposal instructions.

The manufacturer respects the environmental law and contributes to the cost of recovery and recycling.

Follow the user manual instructions.

Do not use if you carry a pacemaker.

Do not use if you are pregnant.

This device is intended for domestic use.

Double insulated electrical appliance (>50V).

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EN - GUARANTEE

We provide a 24 months guarantee for products supplied by us, from the date of purchase.

Never attempt to repair or modify the device yourself, or by a repairman not authorized by SPORT-ELEC®. The guarantee becomes null and void if repairs have been carried out by non-authorized persons or if the replacement parts used are not from the same manufacturer.

Similarly, only use compatible accessories manufactured by SPORT-ELEC®.

During the guarantee period we repair, free of charge, manufacturing defects or faulty parts, reserving the right to decide if certain parts need repairing or replacing, or if the device itself must be replaced.

Any damage resulting from improper use (knocks, plugging into a direct current, wrong voltage, ...) does not fall under the guarantee; normal wear and tear does not affect the proper functioning or value of the device.

The guarantee is only valid if the date of purchase and the stamp and signature of the retailer appear on the guarantee certificate, or upon presentation of the till receipt. (See «guarantee certificate» at the end of this manual).

No other claims fall under the guarantee, unless otherwise provided by law.

Accessories guarantee : The electrodes are designed for personal use. For reasons of hygiene, there will be no trade-in nor exchange.

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