multisportpro

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
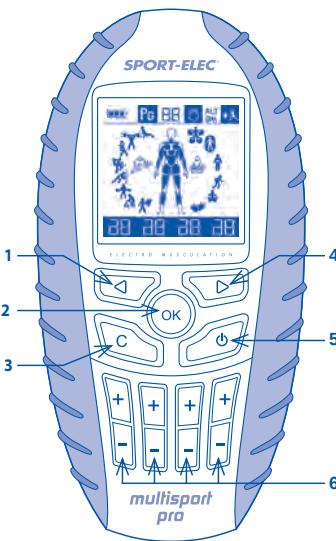
MULTISPORTPRO (4 CHANNELS) DESCRIPTION OF THE UNIT

- 1 Left key : Back or select another choice
- 2 OK key : Validation after each selection
- 3 C key (clear) : Back after validating
- 4 Right key : Choice selection key (programs, muscular zones.)
- 5 ON/OFF key : Turns the unit On and Off / emergency stop
- 6 4 Power key :
+ : Increase power / level 1 to 30
- : Decrease power / level 30 to 1

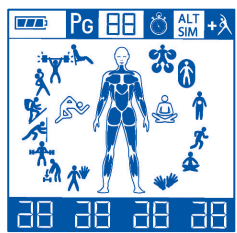DESCRIPTION OF THE SCREEN

- Battery level
- Displays the program number chosen
- Displays the countdown
- ALT** Alternate muscles option (choice between programs PG01 to PG08). Train 2 muscle groups alternately.
- SIM** Simultaneous muscles option (default choice). Train 2 muscle groups simultaneously.
- Enable/disable warm-up phase
- Sport program
- Health program

PROGRAMS

SPORT PROGRAMS

SKIN PREPARATION

Lowers the impedance of the skin barrier. Prepares your skin for the currents emitted by the programs.
Duration : 2 min active phases

RESTORE MUSCLE VOLUME

This program prevents and fights loss of muscle volume resulting usually from a lack of use of a limb following a period of immobilisation.
Duration : 40 min active phase + 6 min recovery

REINFORCE MUSCLE VOLUME

Apply this program only on muscles with a normal volume. PG04 and PG05 are used to develop muscle strength.
Duration : 20 min active phase + 6 min recovery

MUSCLE RESISTANCE

The stimulation frequencies aim at increasing the ability to sustain major muscle efforts.
Duration : 30 min active phase + 6 min recovery

MUSCLE ENDURANCE

After using PG04 to acquire the ability to sustain major efforts, use program PG05 to extend high intensity muscle efforts over time by maximum stimulation of slow fibres.
Duration : 30 min active phase + 6 min recovery

MUSCLE FITNESS

Use this program to maintain the level of muscle endurance, resistance, and volume acquired.
Duration : 30 min active phase + 6 min recovery

EXPLOSIVE FORCE

This program is a logical extension of PG03 and PG04. It allows reaching maximum muscle force by stimulating very high speed fibres, with maximum contraction intensity.
Duration : 30 min active phase + 6 min recovery

RECOVERY

Using a frequency range between 0.5 and 8 Hz, this program ensures muscles recover faster through relaxing, endorphin-based effects, and increases blood flow.
Duration : 20 min active phase

HEALTH PROGRAMS

MASSAGE

This program uses frequencies modulated between 3 and 10 Hz for optimum massaging effect.
Duration : 20 min active phase.

MUSCLE RELAXING

The frequency range between 0.25 and 3.5 Hz of this program accelerates muscle relaxing.
Duration : 20 min active phase

ACTIVE RECOVERY

Use this program immediately after performing intense efforts. All frequencies generated by PG11 maintain muscle work after effort, enhancing blood flow and the release of endorphins. Each of these frequency sequences is alternated with a relaxing effect.
Duration : 20 min active phase

TENS

This program operates by cross-skin nerve stimulation, delivering a range of pulse frequencies in the local area where the electrodes are placed.
Duration : 20 min active phase

ENDORPHINIC

This program relieves muscle tensions effectively.
Duration : 20 min active phase

VASCULARIZATION

This program relieves numb legs sensation.
Duration : 26 min active phase

BEFORE USING THIS UNIT

You have just acquired an effective and powerful professional unit.

Take your time to discover how to use it.

- Read the instructions carefully
- Before using it, charge the unit (photos "G" & "H" page 4)
- Test your unit on your quadriceps using the quick test procedure below

OPERATION QUICK TEST

TESTING YOUR UNIT ON YOUR QUADRICEPS

- Take an electrode, moisten it by spreading a few drops of water onto the adhesive gel (photos "A" & "B" page 3).
- Connect it to the first module (with white colored ring).
- Start the unit using the key and press the "OK" button 6 times.
- Increase power on this channel gradually.
- This discovery procedure shows your capacity in using the unit's level of performance.

* THERAPEUTIC APPLICATIONS

- **Private:** to strengthen and / or to maintain muscle strength.
- **Physiotherapists:** to ease muscular tensions.
- **In hospital environment:** to prevent muscular atrophy and to treat pain.

I. CHARGING THE BATTERIES

The unit contains rechargeable batteries. Before using it, charge it during 12 hours by connecting it to mains. Do not use this unit while batteries are charging. When it is not used, charge the batteries for 12 hours every 4 months. When the batteries are empty, the unit switches to standby automatically, protecting them against untimely discharging. To charge the unit, connect the charger supplied with the unit (photos "G" & "H" page 3)

BATTERY CHARGE DISPLAY

When the battery is charging, the battery indicator displays , , then to show the current charging status. The charging time countdown starts, from to . Once charged, the number appears in the left angle of the timer pictogram. The battery pictogram displays . When the charging is complete, unplug the adapter from the electrical outlet to insulate electric source. When the charger is disconnected, the screen switches off. When the battery indicator is empty , this means that it is discharged.

II. STARTING THE UNIT

STEP 1 : DISPLAY - POWER ON

Beware while starting the device (red button)
Push briefly (maximum 1 second) on .
All pictograms appear and blink for 3 seconds, then the front muscles blink.
If you make a long push, your screen flashes and indicates you the totality of the pictograms which it contains.
To stop the device, push briefly on .
Or to return in the menu made 2 brief pushes on .

STEP 2 : SELECTING THE MUSCLE ZONE(S) TO STIMULATE

Choosing the 1st zone :

The screen displays , meaning you can choose a first muscle zone.
- To choose the front muscles, validate with the "OK" key.
- To choose the back muscles, press the key.
- To return to the front muscles, press the key.
If you have chosen the front muscles : the 1st muscle zone at the bottom blinks: quadriceps.
If you have chosen the back muscles : the 1st muscle zone at the bottom blinks: gastrocnemius (calves)
Press the key to move upwards and scroll the different muscle zones.
To go back, press the key.
After selecting the muscle group, validate using th "OK" key

Choosing the 2nd zone :

All pictograms blink and the screen displays 02.

The screen proposes to choose a 2nd muscle zone :

To choose a 2nd muscle zone, validate again either the front muscles or the back muscles, validate "OK".

Choose again another muscle zone, then validate "OK".

If you wish one muscle zone only, press the "◀" key and go to step 3.

The muscle zone(s) selected is (are) displayed.

STEP 3 : SELECTING THE PROGRAM

Programs on the left (SPORT programs) blink.

If you choose the SPORT programs, validate "OK".

To choose the HEALTH programs, press the "▶" key and validate "OK".

Scroll through the programs by pressing the "▶" key.

The screen first displays the 1st lower pictogram while displaying the corresponding program number at the top.

To go back, press the "◀" key.

You have now chosen the muscle zone(s) to be stimulated, and the program.

STEP 4 : CHOOSE THE "WARM-UP PHASE" AND ALT SIM OPTION

(Only for programs P002 to P007).

The screen proposes a warming-up phase before starting the active phase of the program.

The "WARM-UP PHASE" pictogram blinks in the top right corner.

To select it, press "OK", otherwise press the "◀" key, and validate "OK" (it disappears from the screen).

Then the "SIM" pictogram blinks.

To select "ALT", press the "◀" key, to select "SIM", press the "▶" key, then validate "OK".

If you have selected the warm-up phase, "SIM" is displayed when starting the program, as the warm-up phase is only in the simultaneous mode.

The screen is displayed :

- the battery level,
- phase duration (countdown in minutes),
- timer pictogram,
- the "ALT" or "SIM" option and warm-up phase if you have selected them.

The pictogram for the program chosen is lit, the muscle zone(s) chosen blink alternately.

STEP 5 : LAUNCHING THE START - ADJUSTING INTENSITY

Then, press the "+" buttons of the intensity keys matching the wires connected to your electrodes or your belt to start the program.

If the battery is low, do not start the program.

The power level(s) are displayed at the bottom of the screen. The intensities vary during the session according to the program's content.

A long press on the "—" key returns the power rapidly to 0.

STEP 6 : PROGRAM PERFORMANCE :**Use of the "C" (Clear) key :**

During the program, press the "C" key during a phase to set its time to zero.

The next phase starts unless you have reached the end of the program.

In this case, this key allows returning to selecting the 1st muscle zone (as at normal end of program).

Disconnecting the electrodes during the program :

If an electrode is disconnected, the channel concerned is reset to 00 and blinks for 30 sec, before returning to a fixed 00.

During these 30 sec, the electrode(s) can be reconnected, and power restored to the level required.

STEP 7 : END OF PROGRAM - STOPPING

At the end of the session, the screen proposes to start a new session by blinking the front muscles to select the 1st muscle zone.

Restart the starting procedure if you wish to continue on another muscle zone.

Otherwise, press the "⏻" key to stop.

The unit switches to standby after 5 minutes idle.

Recommendations :

The pictograms for upper muscles and for lower muscles are different. Avoid selecting upper muscles and lower muscles, as the unit would give priority to upper muscles, reducing somewhat the yield of work on lower muscles.

III. USING THE UNIT**A. USING THE DEVICE WITH THE ELECTRODES**

This function allows you to use the adhesive electrodes by connecting them directly to the modules.

You can use the device with 1, 2, 3 or 4 modules.

GETTING STARTED

Remove the plastic backing sheet from the electrodes (photos "A" page 3).

Moisten them by spreading a few drops of water onto the adhesive gel using your fingertips and wait 2 minutes (photo 1-B p. 3).

POSITIONING OF THE ELECTRODES

Fix the electrodes to the module(s) using the snap fasteners (photo "C" page 32).

Position the electrode(s) onto your body.

NB: The electrodes are for personal use. For hygiene reasons, they must always be used by the same person.

STARTING THE PROGRAM:

Start your device by following the «Starting the device» procedure.

Contractions should be firm but must never cause any pain.

If the electrodes cause you any discomfort: reduce the power levels, move the electrode(s) and then gradually increase the power.

Also ensure that the electrodes are sufficiently moistened.

Now that the settings have been optimized, let your device get to work for the full duration of the program.

END OF PROGRAM:

Once the program has finished, switch off the device by pressing the «ON/OFF» button, then disconnect the modules.

Gently remove the electrodes from your skin.

Moisten them slightly with a few drops of water and replace them carefully onto their transparent plastic backing sheet.

Keep the adhesive electrodes in a plastic bag in a cool place (for example, a separate compartment in your refrigerator).

B. USING THE DEVICE WITH THE BELT

The belt has 2 pairs of snap fasteners onto which 2 wireless electrodes specially designed by SPORT-ELEC® are clipped.

To obtain new electrodes, contact your usual retailer or order them from our website www.sport-elec.com.

WARNING!

Never use the belt without attaching the 2 electrodes provided.

Only use electrodes supplied by SPORT-ELEC®.

PRESENTATION OF THE BELT:

See photos "E" and "F" page 4:

- 1 = Snap fasteners
- 2 = Wire storage pouch
- 3 = Velcro strip
- 4 = Belt extension for larger sizes

CONNECTION AND POSITIONING OF THE BELT.

- ▶ Check that your device is switched off.
- ▶ Open out the belt.
- ▶ Fix the electrodes to the inside of the belt using the snap fasteners.
- ▶ Place the belt around the waist and adjust it using the Velcro strip, so that there is a good contact between the electrodes and the skin. If required, use the belt extension.
- ▶ Once the belt is in position, fix 2 modules onto the snap fasteners.
- ▶ If you experience any tingling, switch off the device and check that the electrode is in contact with the skin. If the problem persists, contact the after-sales service: sav@sport-elec.com.

STARTING THE PROGRAM WITH THE BELT

Once your device is connected, make yourself comfortable and switch it on following the «Starting the device» procedure.

You can carry out your normal activities while using the device.

END OF PROGRAM

Once the program has finished, switch off the device and unplug it from the belt.
Remove the electrodes and moisten them slightly with a few drops of water before replacing them onto their transparent plastic backing sheets.

IV. RECOMMENDATIONS OF USE

The unit is recommended for physiotherapists for functional re-education.

- ▶ You can restart a program's cycle on another muscle zone or change the program while remaining on the same muscle zone, without exceeding however 40 min use per muscle zone and per day.
- ▶ Clean the unit and its cables with a cloth dampened with a disinfecting solution (Mercryl type) diluted in water at 1/10th.
- ▶ The adhesive electrodes can be rehydrated before and after use with a disinfecting agent (e.g. Hexamedine).
- ▶ The adhesive electrodes must be repositioned on their support, and stored in a fresh storage between +5°C and +10°C.
- ▶ The adhesive electrodes have a lifetime of about 40 uses, you can order them directly from the online shop www.sport-elec.com, or from your reseller.

PRECAUTIONS OF USE :

- ▶ Do not use the unit on your face
 - ▶ Do not leave within reach of children
- Ask for advice from your physician :**
- ▶ if you have electronic or metallic implants (pins, etc.)
 - ▶ in case of skin condition (wounds,...)

TROUBLE SHOOTING

If your device doesn't work properly, do not use it and contact your retailer.

RECOMMENDATIONS :

- ▶ Please apply regulations enforced in your country when disposing your product at its end of life.
- ▶ Therefore, please discard it in a location planned to this effect, ensuring it is disposed in an environment-friendly manner.

CONTRA-INDICATIONS :

- ▶ **Do not use SPORT-ELEC® :**
- ▶ on the heart area
- ▶ If you carry a pacemaker
- ▶ If you are pregnant
- ▶ during first 6 weeks after baby birth
- ▶ In case of neuromuscular diseases
- ▶ In case of hemorrhagic diseases
- ▶ On the face and skull

WARNINGS :

- ▶ Keep away from children or people with mental disabilities. The device contains small pieces that can be swallowed.
- ▶ In order to prevent any risk of strangulation, please keep the electrodes wires away from children.
- ▶ If the device is not likely to be used for some time, the batteries should be removed
- ▶ Multisportpro is meant for personal use. For reasons of hygiene, the electrode should only be used on one person.
- ▶ Do not use the device by driving or by working on a machine.
- ▶ Do not use this electro stimulation unit in a damp environment, a bathroom, a sauna, or close to electric machinery connected to mains and earthed, or even piping earthed.
- ▶ Use this device in a clean place (without dust nor dirt.).
- ▶ The simultaneous connection of a patient to a high frequency surgical unit may cause burns at the points of contact of the stimulator's electrodes, and the stimulator may be damaged.
- ▶ Operation in close proximity of a short wave therapy unit may cause instability in the stimulator's output power.
- ▶ Do not, under any circumstances, attempt to open or repair the Multisportpro yourself! Only have repairs carried out by the customer service department.

RECOMMENDATIONS FOR MAINTAINING AND CLEANING THE BELT :

- IMPORTANT : NEVER wash in machine and under running water. The dryer and ironing facilities are forbidden.**
- ▶ Do not dry-clean
 - ▶ Do not expose to bleach
 - ▶ Clean the belt using a cloth dampened with a detergent, then wipe with a dry cloth.
 - ▶ Clean the unit with a tissue dampened with a disinfecting solution (Mercryl type) diluted with water at 1/10th.

V. TECHNICAL SPECIFICATIONS

UNIT :

Class Ila medical device in accordance with Directive 93/42/CEE	
Software version.....	E
Type of current.....	Dual phase
Number of programs.....	8 "SPORT" programs - 6 "HEALTH" programs
Number of program variations (or phases).....	344
Number of adjustable independent outputs.....	4
Maximum intensity on a 500 Ω load/channel.....	100 mA / 500 Ω / channel
Frequency range.....	0,25 to 160 Hz
Pulse width range.....	70 to 360 μ s
Power supply.....	4,8 V - 1300 mAh - Ni-MH Rechargeable battery
Mains supply.....	THA239025 : 230VAC 50Hz 100mA - 9VAC 250mA RSS1002-05S110-W2B-B : 100-240VAC 50/60Hz 0.18A - 1,2VDC 250mA
	The adapter is part of the equipment.
	Only use mains power after reticulation by the power unit.
Unit dimensions and weight.....	80 x 160 x 35 mm - 340 gr
Automatic stop at end of each program.....	Yes
Activation safety.....	100%
Temperature for stocking / Relative humidity.....	0°C to + 45°C / 10% to 90%
Temperature for use / Relative humidity.....	5-40°C / 20-65%
Caution !.....	Read the manual thoroughly
Guarantee of the unit / Lifecycle.....	2 years

IP22

RoHS

BELT

References.....	
Size (unisex).....	from 34 to 60
Length of the unfolded belt.....	126 cm
Extension lead.....	~35 cm
Fabric.....	polyester

ADHESIVE ELECTRODES:

Class I medical device (Directive 93/42/CEE)
Adhesive, hydrophilic and hypoallergenic electrodes
References.....EASF110x71
Contact area.....67,5 x 47 mm (x2)
Estimated life expectancy.....40 uses
Please order new compatible adhesive electrodes on www.sport-elec.com.

VI. GUARANTEE

- ▶ We provide a 24 month guarantee for products supplied by us, from the date of purchase.
- ▶ The belt and adhesive electrodes are for personal use.
- ▶ For hygiene reasons, they can therefore neither be returned nor exchanged.
- ▶ The life expectancy of the adhesive electrodes is estimated at approximately 40 uses.
- ▶ Never attempt to repair or modify the device yourself, or by a repairman not authorized by SPORT-ELEC®.
- ▶ The guarantee becomes null and void if repairs have been carried out by non-authorized persons or if the replacement parts used are not from the same manufacturer.
- ▶ Similarly, only use compatible accessories manufactured by SPORT-ELEC®.
- ▶ During the guarantee period we repair, free of charge, manufacturing defects or faulty parts, reserving the right to decide if certain parts need repairing or replacing, or if the device itself must be replaced.
- ▶ Any damage resulting from improper use (knocks, plugging into a direct current, wrong voltage, ...) does not fall under the

CUSTOMER SUPPORT: +33 (0)2 32 96 50 50

guarantee; normal wear and tear does not affect the proper functioning or value of the device.

- The guarantee is only valid if the date of purchase and the stamp and signature of the retailer appear on the guarantee certificate, or upon presentation of the till receipt. (See «guarantee certificate» at the end of this manual
- No other claims fall under the guarantee, unless otherwise provided by law.

ACCESSORIES GUARANTEE :

The belt, cream, and electrodes are designed for personal use. For reasons of hygiene, there will be no trade-in nor exchange.

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EN - WARRANTY

This appliance is guaranteed for 24 months from the date of purchase. During the period of the guarantee, the distributor will take responsibility for defects occur, the appliance will be repaired or exchanged as the distributor deems necessary. Damage arising from improper use or normal wear and defects, which do not materially affect the performance of the appliance, is not covered under this guarantee. The guarantee is not valid if unauthorized repairs have been attempted or if replacement parts used are not approved by the supplier. The guarantee is valid only if it is stamped and signed by the retailer at the time of the purchase. For all claims made during the period of the guarantee, send the appliance and the warranty card to your retailer. All other claims are excluded from the guarantee unless specifically prohibited by law.

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