Tensiomètre

MODE D'EMPLOI

réf : **RTW-8010A**

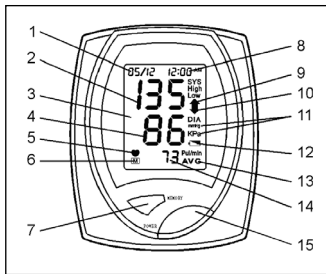
FR

GB

D

ES

NL

SCHEMA

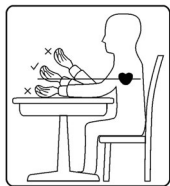
Figure 4

Figure 5

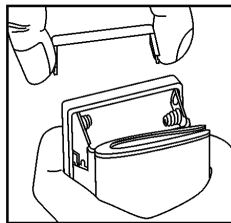
Figure 1

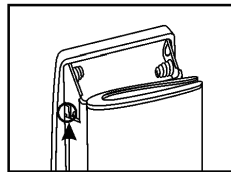
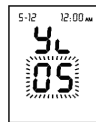
Figure 2

Figure 3

1

2

3

4

5

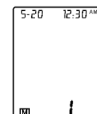
6

7

8

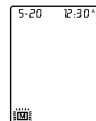
9

10

11

12

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1. INTRODUCTION

Thank you for purchasing this Wrist Blood Pressure Monitor (BPM). This device is designed for individuals age 18 and above, for home use only. Please read these instructions carefully before measuring.

1.1. Features

- One touch. Accurate result in 20 seconds.
- Comfortable automatic inflation and deflation.
- Memories for reading up to 352 sets, with date/ time and automatically calculated average.
- Automatic turn-off after idle 3 minutes.

1.2. Contraindications

- Do not use the device on infants or persons who cannot communicate, for it might lead to serious accidents.
- The device may show some reading error in the presence of common arrhythmia, such as arterial or ventricular premature beats or arterial fibrillation.
- Do not use the device for purposes other than measuring blood pressure.
- Suggest using for individual person to avoid germ infecting.

1.3. Precautions

- This unit does NOT serve as a cure of any cardiovascular symptoms or disease. The data measured is only for reference. Always consult your physician to have the results interpreted. Do not rely on self-diagnosis.
- Always follow the operation procedures described in the manual in order to obtain correct results.
- Do not press the power button when the wrist cuff is not wrapped around the wrist.

- Using the device near a mobile phone or microwave oven may cause a reading error.

- Disassembly or modification the unit or wrist cuff may lead to permanent reading error.

Suggestions before measuring blood pressure:

- Avoid eating, smoking, exercising, bathing, and taking alcohol, caffeine, or tea for at least 30 minutes before measurement.
- Avoid urination or bowel movement before measuring.
- Avoid measuring when feeling anxiety or tension.
- Please sit or lie down for at least 5~10 minutes before measuring.

1.4. Knowledge about blood pressure

Human blood pressure naturally increases after reaching middle age. This symptom is a result of continuous aging of the blood vessel. Further causes include obesity, lack of exercise, and cholesterol (LDL) adhering to the blood vessels, making them lose their elasticity. Rising of blood pressure accelerates hardening of the arteries, and coronary infarction. This is the reason why we need to know our routine blood pressure trend so as to see whether our body is in good condition or not. The blood pressure fluctuates constantly throughout our daily life and is an essential element in our daily health control.

The following diagram is for your reference.

	Systolic	Diastolic
Hypertension	>160 mmHg	>95 mmHg
Borderline	140 à 160 mmHg	90 à 95 mmHg
Normal	90 à 140 mmHg	< 90mmHg
Hypotension	< 90 mmHg	

2. DEVICE DESCRIPTION

- 1.Date
- 2.Systolic Pressure
- 3.LCD DISPLAY
- 4.Diastolic Pressure
- 5.Pulse symbol
- 6.Memory symbol
- 7.Memory button: To set the Date/ Time and to view the stored measurements.
- 8.Time
- 9.Inflation
- 10.Deflation
- 11.Measurement unit
- 12.Battery symbol
- 13.Average
- 14.Pulse rate
- 15.Power button: Press to start inflating the cuff and measuring the blood pressure & pulse rate. Press again to turn off the device.

3. PREPARATIONS BEFORE USE

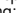
3.1.Battery installation / replacement

- 1.Press the sides of the battery cover to remove it. (fig. 1)
- 2.Install or replace 2 "AAA" size alkaline batteries. Make sure the + (positive) and - (negative) marks match those indicated in the battery compartment.
- 3.Replace the battery cover.

Note: For best performance and longest life, we recommend using alkaline batteries.

Do not mix difference type, date or brand-name batteries.

This device features 2-setup battery warning:

a.Battery low warning:  battery indicator appears on LCD display. It means the battery is running low. You can still make about accurate measurements.

b.Battery dead warning:  when the battery indicator flashes, the battery is dead. Please change immediately.

3.2.Date, Time, and Unit Setting

Note: Be sure to turn off the unit before setting Date/Time.

1.Slide the battery cover off. The "SET" button is the black button at the side of the unit. (Figure 2) Press the "SET" button with the tip of a pen to access the date/time setting menu.

2.You will see "Yr 05" on the screen, with "05" flashing. (1-2 p3)

3.Press the "memory" button to set the number representing the year. (when holding the "memory" button for about 2 seconds, the number will run in faster speed.)

4.Press "set" button again to set the year. Now the number representing the "month" starts to flash.

5.Press the "memory" button to set the number representing the month.

6.Press "set" button again to set the month. Now the number representing the "day" starts to flash.

7.In the same manner, set the "hour" (shown in AM/PM), "minute", and "unit".

8.After setting the unit, press the "set" button again to turn off the unit.

Note: After battery replacement, always confirm the time before using the blood pressure monitor.

3.3. How to apply the wrist cuff

1.Hold your left arm in front of you with palm facing up.

2.Apply the cuff so that the monitor is on the inside of your wrist with the display in easy view.

3. Wrap the cuff comfortably around your wrist. The top and bottom edges of the cuff should be tightened evenly around your wrist. The edge of the cuff should be approximately 1/4 to 1/2 inch from below your palm.

4. When the cuff is positioned correctly, secure it by pressing the pile material firmly against the hook material.

Note: Blood pressure varies from two hands naturally. Therefore please measure your blood pressure on the same wrist to ensure comparability of readings.

There is a difference between blood pressure values taken at the wrist and upper arm, but generally blood pressure values at the wrist and upper arm should display the same pattern of change.

3.4. Proper measurement position

It is extremely important that the cuff be at the same level of the heart. If the wrist is relatively lower (higher) than the heart, the blood pressure measured will be higher (lower) than the actual value. A 15cm difference in height may result in an error of 10 mmHg.

Do measurement while sitting:

1. Place your elbow on a table, palm facing up, Reset your wrist on the case to make sure the pressure cuff is at the same level as the heart. (fig4 p2)

2. Press "power" button and relax.

4. HOW TO TAKE A READING

Please follow directions in "Suggestions before Measuring Blood Pressure" and "How to Apply the wrist cuff" before taking a reading.

1. Apply the wrist cuff before pressing "power" (fig5 p2)

2. Press the "power" button. All the LCD display symbols will appear. (3 p3) this is the initial LCD test. When the monitor completes necessary preparations before measurement, you will see the last rea-

ding taken on this unit. (4 p3)

3. After 3 seconds. The cuff automatically inflates to approximately 160 mmHg. (this is approximately 40mmHg higher than the average systolic blood pressure reading). The increasing symbol "▲" will appear during inflation. (5 p3)

4. After reaching your correct inflation level, the cuff begins to deflate automatically. As the cuff deflates, decreasing symbol "▼" appears on the display. The pulse symbol "///" flashes at the onset of oscillation signals. (6 p3)

* If during the measurement the monitor determines that a higher-pressure value is required, it will pause deflation and re-inflate.

5. The monitor exhausts the residual air when the measurement is complete. The pulse symbol stops flashing. Your blood pressure and pulse appear on the LCD display. (7 p3)

6. Reading taken will be displayed before the power is turned off (or after 3 min).

Note: Please take a 5-10 minute break before taking another measurement. Waiting allows the engorged blood vessels to return to normal status. The break can be longer, depending on your physiological condition.

5. HOW TO USE THE MEMORY FEATURES

5.1 Memory mode

The blood pressure and pulse rate readings measured are automatically stored in the memory. The unit stores the most recent 352 sets of readings. When more than 352 measurements were taken, the earliest set will be replaced by the most recent. Use the memory function to recall previous readings.

1. Press the "memory" button.

2. The first data shown is the average of all data stored (8 p3), including systolic and diastolic blood pressure, and pulse.

3. Press "memory" button again, the index "1" will be displayed, followed by the last reading taken. (figure 10) When holding the "memory" button 2 seconds, the number will run faster.

4. To recall the readings measured previously, press "memory" button again. The values will be displayed in the order of time and date taken.

5.2 How to erase all the readings in memory

Note: Be sure to turn off the unit before erasing all readings.

1. Press and hold "memory" button for 4 seconds.

2. Wait until the memory symbol and "Clr ALL" appear on the LCD display (10 p3)

3. Release "memory" button and all the memories have been erased. The unit will automatically turn off. (11 p3)

Note: All data will be erased. This is no way to erase on single set of readings.

4. If you press "memory" button again, this is the screen you will see when there is no data in the memory.

6. CARING FOR YOUR METER

6.1 Maintenance

Storage

- Batteries may leak chemicals that damage electronic parts. Please remove the batteries if the monitor won't be used for an extended period of time.

- Always use the storage box provided with the device to transport or store the unit.

- Store the unit in a dry and cool place within the storage condition range. The device shelf life is 3 years in normal condition except for battery. Do not operate or store the device in direct sunlight, high temperature, humid, dusty, or corrosive environment.

- Please refer to "specification" for the proper storage and operation environment.

- Do not drop this unit. Avoid strong impact.

- If you are not going to use the device for an extended period, remove the batteries from the unit.

Cleaning

- Do not use benzene, gasoline, or any alcoholic-base or solvent agent to clean the device.

- To clean the unit, wipe with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft and dry cloth. Do not flush with or immerse in water.

- The cuff can be cleaned by wiping with moistened cloth and soap. Do not immerse in water.

- Do not twist any tubing. Refrain from stretching, bending or tugging on the wrist cuff. Do not flip the cuff inside out.

- Do not separate the cuff from the unit, disassemble the device, or try to repair the unit by yourself. If any problem occurs, refer to the distributor.

6.2 Error message & troubleshooting

The unit is manufactured through the complete quality assurance system. If there are performance problems, please check the troubleshooting table for problems. You can fix at home without tools.

Symptoms/indicators, cause and counter-measure

No indicator/symbol on the LCD although batteries are installed ;

Batteries ran out or were not installed properly : Check and correct the battery polarities. Remove the batteries and wait 10 seconds before replacing the batteries.

ERR 00; No pulse, or detected pulse not enough: Refit cuff tightly, repeat measurement as shown in "how to apply the wrist cuff"

ERR 01; Cuff serious leakage or inflation too slow: Replace the batteries first. If pro

lem remains, have the unit checked at a service center.

ERR 02; Cannot figure out systolic: Refit cuff tightly, repeat measurement as shown in "how to apply the wrist cuff"

ERR 03; Cuff pressure is too high: Refit cuff tightly, repeat measurement as shown in "how to apply the wrist cuff"

ERR 04; Cannot figure out diastolic or it's beyond normal range: Refit cuff tightly, repeat measurement as shown in "how to apply the wrist cuff"

ERR 05; Deflation too fast: Have the unit checked at a service center

ERR 06; Deflation too slow: Have the unit checked at a service center.

ERR 07; Systolic beyond normal range: Refit cuff tightly, repeat measurement as shown in "how to apply the wrist cuff"

ERR 08 Mean pressure beyond the legal range: Refit cuff tightly, repeat measurement as shown in "how to apply the wrist cuff"

ERR 09; Cuff pressure not enough even if the highest value is pumped: Refit cuff tightly, repeat measurement as shown in "how to apply the wrist cuff"

ERR 330; Cuff pressure is higher than 330 mmHg: Refit cuff tightly, repeat measurement as shown in "how to apply the wrist cuff"

6.3 Specifications

- Display: LCD digital display
- Measurement Method: Oscillometric method
- Measurement Range
Pressure : 0~300mmHg
Pulse rate : 40~199 / minute
- Accuracy / Calibration
Pressure : 3mmHg of reading
Pulse rate : 4% of reading
- Memory Capacity: 352 sets of reading with date and time
- Inflation: Automatic inflation

- Deflation: Electric valve
 - Pressure Release: Automatic exhaust valve
 - Pressure Detection: Piezo-Resistive silicon pressure transducer
 - Power supply: 2x AAA 1.5V alkaline batteries
 - Operation Environment: +10°C ~ +40°C / 30% ~ 85% R.H. maximum
 - Storage Environment: -20°C ~ +60°C / 10% ~ 95% R.H. maximum
 - Battery Life: Approx. 200 uses (alkaline batteries)
 - Cuff Size: Wrist Circumference approx. 135~195mm (5-1/4 ~ 7-3/4 inch)
 - Dimensions: 76mm x 64mm x 29mm
 - Weight: 132g (with batteries)
 - Accessories: Storage Case, Instruction Manual
- *Please understand that specification may be changed without prior notice.

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