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- 13 -

ELECTRO-BODYBUILDING

What is the purpose of electro-bodybuilding?

The use of electro-bodybuilding by the general public is justified by its efficiency in restoring and building up muscle, particularly in the areas of the abdomen, the buttocks, the thighs and the quadriceps.

Electro-bodybuilding covers a very wide field, including all types of sporting activities.

Apart from building up muscle volume, strength and resistance, the use of electro-bodybuilding in sports is an excellent means of allowing the metabolism of the muscles to recover after intense effort.

In addition, electro-bodybuilding is recommended for persons wishing to take up a sporting activity or wanting to build up their muscles after an accident or an operation. The unused and untrained muscles may be brought back into action in a very progressive manner, thus avoiding any muscular damage.

Depending on our daily activities, some of your muscles are more or less developed whereas other, rarely used muscles atrophy.

We have more than 600 muscles in our body. Using all of the muscular fibres within a muscle requires intense and prolonged physical effort. With electro-bodybuilding, this work is achieved without effort.

Our muscles play an important role in the condition and harmony of our body as a whole.

Electro-bodybuilding with **SPORT-ELEC** allows you to work your muscles, whatever your physiology.

And just like many top sportspersons, your **SPORT-ELEC** means you benefit from high technology which will allow you to have more success in your own sporting discipline.

How does it work?

Electro-bodybuilding is the action that causes a contraction of the muscular fibres.

The natural contraction of body muscles occurs via electrical pulses sent from the central nervous system.

Electro-bodybuilding induces muscular contractions through external electrical pulses (from power generators) via electrodes placed in contact with the skin, over the muscle that one wishes to contract.

There are two separate type of muscle contraction:

- Isometric contraction, which takes place without any movement and without any change in the length of the muscle segment.
- Isotonic contraction, which involves a contraction of the muscle.

Who can use it?

Everyone, whether involved in sports or not, from amateurs to top professionals.

What are the limitations in its use?

The only contraindications concern people fitted with pacemakers and pregnant women (see Precautions for use / Contraindications) and application on areas with electronic implants (e.g. : COCHLEAR implants).

Physiotherapists recommend **SPORT-ELEC EVOLUTION KINÉ** to their patients. **SPORT-ELEC EVOLUTION KINÉ** has 117 programme variations in 8 phases dedicated to muscular recovery, relieving pain and combating muscular atrophy. Thanks to its 2 independent generators, you can work 2 different muscle areas simultaneously and adjust the intensity for each area. For optimum use, we recommend that you carefully read the following information and instructions.

DESCRIPTION OF THE APPARATUS

(See diagram 1, page 7)

- 1 - Programme number
- 2 - "Performance Control" bar-graph
- 3 - Programme selection button
- 4 - Start / Stop button
- 5- Power button, Black channel (A)

- 6 - Power button, Grey channel (B)
- 7 - Wire guide
- 8- Plugs
- 9- 4 body electrodes (89x50 mm)

SPORT-ELEC EVOLUTION KINÉ contains 2 independent power generators (including 2 sets of 2 electrodes, each with a large contact area of 44 cm²). These electrodes ensure perfect diffusion of the power to the muscles that need to be stimulated. **SPORT-ELEC EVOLUTION KINÉ** is a registered trademark. It is supplied in a polished ABS casing. Two sets of wires, 80 cm long, are used to connect the apparatus to the body electrodes (89 x 50 mm).

PRECAUTIONS FOR USE :

- * Do not use **SPORT-ELEC EVOLUTION KINÉ** on your face
- * Keep out of reach of children
- * Consult your doctor:
 - if you have any electronic or metallic implants (joint pins, etc.)
 - if you have any skin disorders (wounds, etc.)

CONTRAINDICATIONS :

- Do not use your **SPORT-ELEC EVOLUTION KINÉ** :
 - * On the frontal cardiac area;
 - * If you wear a cardiac stimulator (pacemaker)
 - * If you are pregnant

WARNING:

Do not use your **SPORT-ELEC EVOLUTION KINÉ** in a humid environment, in the bathroom, sauna or next to electrical devices with mains and earth connections. Simultaneously connecting a patient to a high frequency surgery unit may cause burns at the electrode contact points of the stimulator and the stimulator itself may be damaged. Short wave therapy devices operating in the immediate proximity may cause power instability in the stimulator output.

HOW TO USE YOUR SPORT-ELEC EVOLUTION KINÉ

- 1°** Check that the **SPORT-ELEC EVOLUTION KINÉ** switch is in the "0" position and the power button on minimum (up against the left stop). Open the battery compartment, insert 2 x 9 volt batteries the right way up (+/-) and then close the case.

(see diagram 1 page 7)

- 10 - Spring clip
- 11 - Battery case
- 12 - 9 volt batteries (not supplied)
- 13 - Removable cover

Lifetime of the batteries: 1 month when used for 20 min./day. If not used for long periods, remove the batteries from their casing.

- 2°** Pick up the 4 electrodes and carefully remove the support used to protect each electrode. Moisten the electrodes in turn by spreading several drops of water on the adhesive gel and then leave for 2 minutes (see diagram J, page 7). Place the electrodes on previously washed and dried skin, according to the examples given below. The lifetime of the electrodes depends on how often they are used (on average, around 60 sessions). Connect up the electrodes to the generator.

- 3°** **STARTING A SESSION / SELECTING A PROGRAMME**

a) Check that the 2 Black channel (channel A) and Grey channel (channel B) power buttons are in the minimum positions (as a safety precaution before starting up the apparatus). Turn the switch to "1".

b / CHOOSE YOUR PROGRAMME :

All you have to do is press on the "programme selection" button placed under the screen, and the programmes will scroll :

PG01 : WARMING UP MUSCLES

Your muscles are prepared for effort and are warmed up. You can follow this with a bodybuilding programme or in enjoying your favourite sporting activity.

PG02 : BUILDING UP MUSCULAR RESISTANCE

This programme brings into use the specific fibres required for building up muscular resistance to effort.

PG03 : ACQUIRING MUSCULAR STRENGTH

This programme brings into use the muscular fibres required for building up strength.

PG04 : TENS

Relieves local pain.

PG05 : ACTIVE RECOVERY

This programme should be used immediately after intense physical effort. The series of frequencies generated by this programme allows you to keep the muscles working after physical effort, to increase blood flow and to liberate endorphins. Each of these frequency sequences is alternated with a relaxing effect.

PG06 : ACQUIRING ENDURANCE TO EFFORT

This programme allows you to maintain, over time, high intensity muscular effort by stimulating the slow muscle fibres to the maximum extent.

PG07 : HIGH SPEED STIMULATION OF FIBRES

Facilitates rapid and intense muscular work.

PG08 : ENDORPHINIC

Relieves and reduces diffuse pain.

NB :
To get the most out of your machine, carefully read each page and follow the instructions.

A - Pectorals 4 - Forelimb
1 - Major pectoral
Do not place the electrodes on the frontal cardiac muscle

C - Abdominals 5 - Superior oblique muscle

E - Trapezius 9 - Upper trapezius muscle
14 - Semi tendinosus muscle

G - Legs 13 - Crural biceps muscle
18 - Vastus medialis muscle


H - Thighs 17 - Major adductor muscle
18 - Vastus medialis muscle

Examples of use : A = canal A (back button); B = canal B (grey button)


You can put 2 electrodes on a muscular area then 2 on another (on a skin that has first been washed and dried).

Example of positioning 1

example : right thigh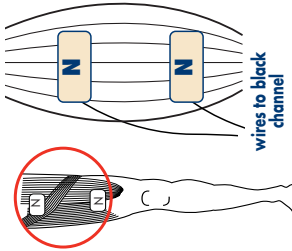

example : left thigh

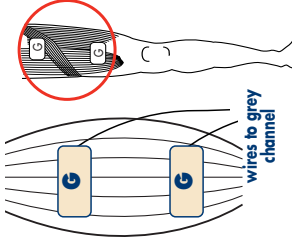
N Connect the electrodes to the black connectors


G Connect the electrodes to the grey connectors

Example of positioning 2


example : right thigh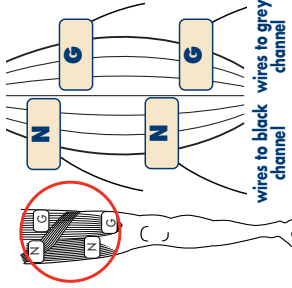

example : left biceps

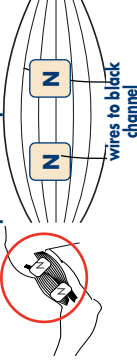example : left biceps

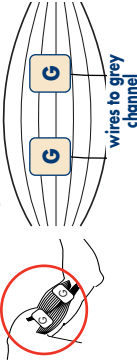4°

PERFORMANCE CONTROL FUNCTION

SPORT-ELEC EVOLUTION KINE is equipped with a performance indicator that allows you to evaluate how much of the total power capacity you are using. This is a function of the length of use, the intensity of the session and the number of generators used.

Example.: If the indicator is on 1, the apparatus is using between 10 and 19 % of its total capacity.

1 : 10 to 19 %	4 : 40 to 49 %	7 : 70 to 79 %
2 : 20 to 29 %	5 : 50 to 59 %	8 : 80 to 89 %
3 : 30 to 39 %	6 : 60 to 69 %	9 : 90 to 100 %

Once the programme has ended, turn the power buttons back to the minimum position and turn the switch to the "0" position. Carefully disconnect and remove the electrodes from the skin, lightly moisten them with several drops of water and carefully put them on their supports. The electrodes should be stored in their original packaging in a cool place (the vegetable compartment of a refrigerator is ideal).

TIPS FOR USE

- You can restart the cycle of one of the programmes on another muscle area without exceeding 40 minutes of use on the same part of the body. If you want to stop a programme that is already underway, you just have to turn the 2 buttons to "0" and start the device up again while selecting a new programme.
- For optimum use, you can draw inspiration from the diagrams in this brochure.
- The casing and cables of your **SPORT-ELEC** may be cleaned with a cloth dampened with disinfectant (such as Mercryl) diluted with water in the ratio 1/10.
- The parts applied to the body, such as the electrodes, may be re-hydrated before and after use with a disinfectant (e.g.: Héxoméline solution).
- The electrodes should be placed back on their supports and kept in a cool place between +5 °C and +10 °C.
- If you do not intend to use your **SPORT-ELEC** for a long time, remove the batteries from the casing.
- The 4 electrodes can be used around 60 times. You can order new ones from your normal retailer or directly from **SPORT-ELEC** at the following address : **SPORT-ELEC 5.A**
BP 35 - Route de Rouen - 27520 Bourgtheroulde, France
TEL : +33 (0)2-32-96-50-50 - FAX : +33 (0)2-32-96-50-59
www.sport-elec.com

DIFFERENT APPLICATIONS FOR THE SPORT-ELEC EVOLUTION KINE PROGRAMMES

TYPE OF CURRENT	SYMPTOMS	PROGRAMMES CONCERNED
Analgesic current through inhibition of segmental sensitivity (localised pain)	Sprains, pulled muscles, tendonitis, neuralgia, sciatica, congestive osteoarthritis, etc.	PG04 : TENS
Analgesic current through the liberation of endorphins (diffuse pain)	Lumbago, back pain, neck pain, osteoarthritis of the rachis, diffuse pain, etc.	PG05 : Active recovery PG08 : Endorphinic
Excitomotory current through a trophic tetanising effect	- Severe stiff neck, spinal contractions, hamstring contractions, etc.	PG01 : Warming up of muscles PG02 : Building up muscular resistance